Vet Scales 60cm x 90cm

210702

INSTRUCTION MANUAL

www.monstershop.co.uk

SPECIFICATION

PRODUCT DESCRIPTION

Our T-MECH Vet Scales is a product of great value to any veterinary practice or surgery! The digital indicator will ensure accurate readings of creatures great and small!

PRODUCT SPECIFICS

MODEL:	XK3190-B1 Digital weighing indicator
ACCURACY:	Class 3 N=3000
SAMPLE SPEED:	10 times/sec
LOADCELL SENSITIVE RANGE:	1.5~3mV / V
DIVISION:	1/2/5/10/20/50 optional
DISPLAY:	6 bits LCD/LED display
KEYPAD:	With 4 mechanical buttons
PC POWER SUPPLY:	3.7V/2000MAH lithium battery 5V charger
OPERATING TEMPERATURE:	-10~40°C
HUMIDITY:	≤85%RH
TRANSPORTING TEMPERATURE:	-25~55°C

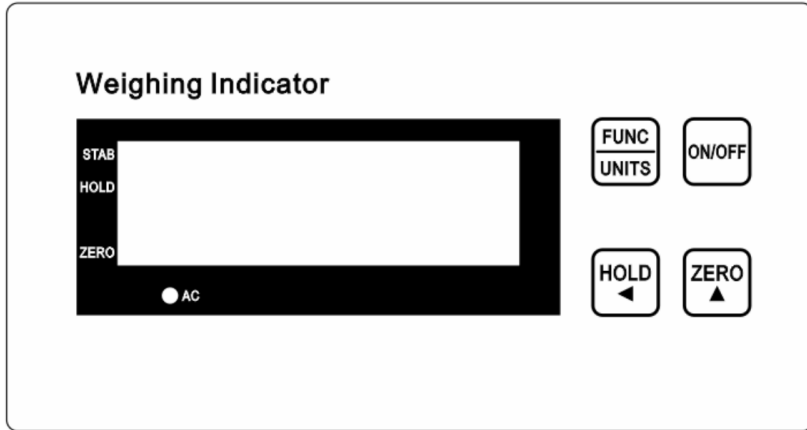
KEYPAD INSTRUCTIONS

Keys	Function
ON/OFF	Turn the device on and off.
FUNC/UNITS	Long press this key to enter settings. Short press this key to switch between kg/lb.
HOLD	Long press to hold the weight value. Press this to shift when calibrating. Press this key to add value 1 into the system settings.
ZERO	When in weighing mode, press this key to bring the weight to zero. During calibration, press this key to add value 1.

Monster Group UK Limited's liability will be limited to the commercial value of the product only.


SPECIFICATION

DIAGRAM

CONNECTION INTERFACE

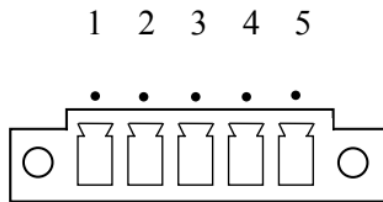- The loadcell is connected by a 9-pin plug holder by default. Some indicators use a 5-pin aviation connector. The following diagram shows the meaning of each pin respectively.

1、E+ 2、E- 3、S+ 4、S- 5、SHIELD

5pin air port load cell connection method

- When using a 4-pin shielded cable, the indicator has no long line compensation by default. The long line compensation is optional.
- The connection between loadcell and indicator must be reliable, the loadcell's shielded cable must be connected to the ground reliably.

USER GUIDE

GENERAL INSTRUCTIONS

- **Zero** - The ZERO function allows you to remove unwanted weight from the indicator display. The display will show “0” and the zero point light will turn on.
- **Unit Switch** - The UNIT function allows you to toggle between kg and lb readings. The unit of measurement will be indicated on the display.
- **Weight Holding** - The HOLD function allows you to hold on the current weight value in weighing mode. The HOLD light will turn on.

SYSTEM SETTINGS

Long press the FUNC key to enter user’s settings mode.

Press HOLD to modify parameter values.

Press ZERO to enter the next parameter settings.

Press FUNC again to save and exit once the changes have been made.

Parameter	Function	1	2	3	4	5	6	7
P1	Backlight setting	Turn off backlight	Auto backlight	Backlight always on	-	-	-	-
P2	Zero point tracking	0.5e	1.0e	1.5e	2.0e	2.5e	3.0e	5.0e
P3	Filter depth	Fast	Middle	Slow	-	-	-	-
P4	Stabl time	Fast	Middle	Slow	-	-	-	-
P5	Filtering amplitude	Low	Middle	High	-	-	-	-

SAFETY ADVICE

SAFE WORKING PRACTICE

WARNING!

The loadcell and indicator are equipment sensitive to static electricity. Anti-static precautions must be taken when in use.

- Before using the Vet Scales and Indicator ensure the scale is on a flat surface. An un-level surface could result in inaccurate readings.
- The loadcell's accuracy and structural integrity can be compromised if mistreated, including from welding near the loadcell. Do not subject the scales to welding or any other similar activity that may negatively affect the loadcell.
- Observe the permissible ambient temperatures for use in the "Product Specifics" section.
- This product is not waterproof. It is not recommended to use the scale in humid, damp or wet environments.
- Do not exceed the maximum values specified in the "Product Specifics" section.
- To prolong the life of your product, do not place the loadcell and indicator in direct sunlight or dusty areas.
- Do not use any harsh chemicals to clean this product. Avoid using water or liquids which may lead to an electric shock.

USE OF BATTERY

- The indicator uses a Mini-USB charging interface. Charging voltage is 5V.
- When the indicator is displaying "Lob", this means that the battery power is too low. Turn OFF the indicator and charge the battery immediately.
- When using the indicator for the first time, please charge the battery first. It is recommended to charge the battery for more than 24 hours to ensure that the battery is fully charged.
- In order to avoid battery life damage, it is recommended to charge the indicator immediately once the battery is in "Lob" state. It is recommended to charge the indicator for more than 12 hours each time. If the indicator is not being used for a long time, the battery should be charged every 2 months for more than 12 hours to avoid issues with battery life.

USER GUIDE

CALIBRATION INSTRUCTIONS

Before calibration, open the lead seal behind the indicator and unscrew the calibration screw. In the shutdown state, long press the FUNC key and hold it down while also pressing the ON/OFF key, to start the indicator. When the weight window displays U X, release the keys to enter the calibration procedure. The calibration steps are as follows:

Press ZERO to select calibration unit:

Display	Parameter
U X	kg
	lb

Press FUNC to enter division value settings. Press ZERO to select division value:

Display	Parameter
d X	1
	2
	5
	10
	20
	50

Press FUNC to enter decimal point settings menu. Press ZERO to set corresponding decimal point:

Display	Parameter
P X	0
	0.0
	0.00
	0.000

USER GUIDE

CALIBRATION INSTRUCTIONS

Press FUNC to enter max capacity settings menu. Press HOLD to enter capacity value setting. Press HOLD to shift. Press ZERO to add value 1:

Display	Parameter
FULL	Press HOLD to enter capacity value setting
2000.0	Press HOLD to move the digit from left to right. The number will be flashing. Then press ZERO to add 1 value on the flashing number till the corrected number appears

Press FUNC to enter zero point calibration menu:

Display	Parameter
noLoAd	Wait for the stable light to enter next step

Press FUNC to enter the loading calibration settings menu:

Display	Parameter
AdLoAd	Press HOLD to enter calibration value setting
00000.0	The last digit is flashing
200.0	Press HOLD to shift. Press Zero to add value 1. Place the corresponding set value weight onto the scale platform.

Press FUNC to finish set up:

Display	Parameter
End	Calibration setting finish

When the indicator displays “END”, push the calibration switch with the long screw. Place back the lead seal and screw the calibration screw back in place.

TROUBLESHOOTING

RESOLUTION GUIDE

WHAT DO I DO IF THERE ARE MISSING PARTS FROM MY ORDER?

If there appears to be any part missing from your order, contact our friendly and helpful Customer Support Team within 7 days of receipt.

Error	Meaning
Err1	The AD value is too small when calibrated or the loadcell capacity is too large
Err5	There are no heavy weights on the platform in calibration mode
Err7	The loadcell cable connection is wrong, please connect the loadcell cable correctly
Err8	Loadcell signal is unstable
Upper horizontal bar	Overload; Loadcell is not connected or loadcell connection method is wrong
Lower horizontal bar	Underload; Loadcell is not connected or loadcell connection method is wrong
Lob	Battery power is too low, please charge the battery

FOR ALL OTHER ISSUES, PLEASE CONTACT OUR CUSTOMER SUPPORT TEAM.

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Monster Group (UK) is one of the UK's fastest growing online retailers. We supply a diverse range of products which offer exceptional value for money. We strive to provide the best possible customer experience with free UK delivery across all our products and a Monster Guarantee for 100% satisfaction.

For information regarding this device or other products from our Monster divisions please contact Sales.

SALES

TEL: 01347 878 888
EMAIL: hello@monstershop.co.uk

For queries about this device, warranty, returns or reporting faults please contact Support.

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