



Electric Beauty Massage Chair

24418



MonsterShop

Try **Monster** - Buy **Monster** - Be **Monster**

Specification

Product Description

Thank you for purchasing this MonsterShop Electric Beauty Massage Chair.

This elegant and stylish faux leather Electric Beauty Massage Chair is designed to suit a vast array of professional environments and industries. The sleek design is a perfect addition to hair and beauty salons, cosmetic salons, tattoo parlours, barber shops, and dental and doctor's surgeries, and can even be used for home use.

The manual tilting back rest and foot rest allows a fully adjustable chair operation, and the removable head cushion in place of a face hole allows all manner of medical treatment and massage therapy to be provided with ease and maximum client comfort.

The 10cm high-density sponge chair is separated into three sections, ensuring the electric height adjuster operates smoothly. The PVC faux leather material is easily wiped and long-lasting, while the FREE Massage Stones allow you to be up and running promptly.

WARNING: The maximum load capacity of this chair is 150 KG. Do not exceed this limit under any circumstances.

We at Monster Group UK understand this is a new and exciting purchase, and although it is tempting to get stuck in please carefully read these instructions before you start using the machine.

Please refer to and read Safety Advice, Safe Working Practice to ensure prevention of injury or damage to the device before starting.

Item Contents

- 1 x MonsterShop Electric Beauty Massage Chair (Cream)
- 1 x Assembly Pack of Screws
- 1 x Set of FREE Massage Stones
- 1 x Standard UK Plug fitted
- 1 x EU Adaptor

Specification

Technical Information

Voltage: 220V

Frequency: 50Hz

Power: 75W

Product Specifics

Dimensions: (H x W x D) 60-82cm x 73cm x 188cm

Weight: 45 KG

Material: PVC Leather

Cushion depth: 10cm

Capacity: 150 KG (optimum weight max. 100 KG)

Structure: Removable arm support and head cushion, manually adjustable foot and head rest, electric motor to adjust height 60cm-82cm

Monster Guarantee

If you wish to return a product in perfect working order, we provide a 30 day returns policy as long as the item is unopened and in a resalable condition.

A 12 month warranty applies to all of our electrical products; we will cover the cost of labour and parts. Our policy is to try and repair the item before arranging an exchange or refund.

If for any reason a part is missing please get in touch with us within 7 days on receipt of your order. You can contact our friendly and helpful Customer Support Team via email or call. For full terms and conditions contact our Support Department via the details on the Contact Us page.

Monster Group UK Limited's liability will be limited to the commercial value of the product only.

Safety Advice

Safe Working Practice

Please read through the safe working practice to ensure prevention of injury or damage to the device.

Do not use this product for anything other than its intended use.

If the product begins to produce excess heat, fully turn it off immediately and the product to cool for no less than one hour before using again.

Ensure the product is completely turned off and unplugged when not in use.

Do not leave the product unattended when it is operational.

Do not allow the product to get wet. Do not turn the product on if you have wet clothes, hair, or skin.

Only operate the product on a dry, heatproof, and even surface.



Specification

Product Features



1: Controller: Allows you to raise and lower the chair electronically.

2: Seat: High-density, 10cm depth sponge for extra comfort.

3: Removable Headrest: Creates a face hole for massages and therapeutic practices.

4: Removable Arm Rests: Flexible design dependent on client needs.

5: Motor: Powers the chair's height adjustment.

User Guide

Assembly Parts

A: 1 x Frame



B: 1 x Seat



C: 2 x Armrest



D: 1 x Footrest Stabiliser



E: 1 x Backrest Stabiliser



F: 4 x Seat Screws



G: 4 x Feet



User Guide

Assembly Instructions

Ensure assembly is undertaken on a solid and level surface.

1) Place the Seat (B) onto the Frame (A) and use 4 x Seat Screws (F) to attach, as in Image 1.

IMAGE 1


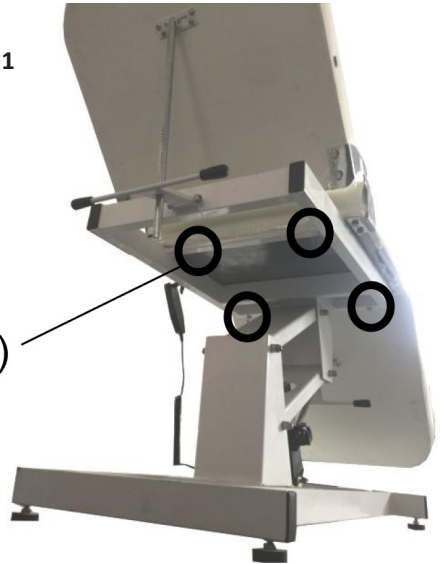


IMAGE 2

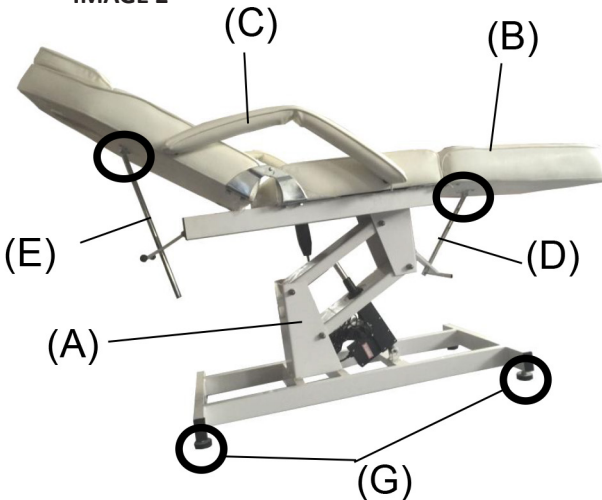

2) Connect the Footrest Stabiliser (D) to the footrest.

3) Connect the Backrest Stabiliser (E) to the backrest.

4) Carefully place the 4 x Feet (G) to the base of the frame. See Image 2.

Your Electric Beauty Massage Chair is now operational.

Maintenance

The Electric Beauty Massage Chair should not need to undergo any general maintenance after assembly.

To protect the material, you should clean the surface regularly. Do not use harsh cleaning agents as this will damage the material.

Use a damp cloth and wipe down after every use.

It must again be stressed that the maximum load capacity is 150KG. The chair should never be overloaded at any time.

Troubleshooting

Resolution Guide

Please read through the guide below if you have any issues or faults with your device. The information covers and resolves the majority of frequently asked questions.

What do I do if...

Q: ... there are parts missing from my order?

A: If there appears to be any part missing from your order, contact our friendly and helpful Customer Support team via the details on the Contact Us page within 7 days of receipt.

Q: ... the motor is not running?

A: Check that the chair is connected to the mains and is switched on. Use another mains socket if necessary. If the problem persists, check that you have not exceeded the 150KG load limit.

Q: ... the parts are loose/the chair wobbles?

A: Check that all the screws are tightly screwed. Check that the stabilisers are still firm. Check that the four feet are still attached.

For all other issues please contact our Customer Support department via the details on the Contact page.

Contact Us

Sales Department

For information regarding this product or other products from our Monster divisions please use the following details below.

Tel: 01347 878888

Email: hello@monstershop.co.uk

Support Department

For queries about this product, warranty, returns or reporting faults please use the following details below.

Tel: 01347 878887

Email: help@monstershop.co.uk

Website

To view our product range and fantastic offers in the MonsterShop division please visit our website;

www.monstershop.co.uk

Address

To visit our office Monster House and view our products, send postal correspondence or return items our address is provided below.

Monster House, Alan Farnaby Way, Sheriff Hutton
Industrial Estate, Sheriff Hutton, York YO60 6PG



Monster Group (UK) is one of the UK's fastest growing online retailers. We supply a diverse range of products which offer exceptional value for money. We strive to provide the best possible customer experience with free UK delivery across all our products and a Monster Guarantee for 100% satisfaction.

Our dedicated customer service team will be more than happy to help with any questions you may have.

Home & Garden
Storage Shelving
Sign & Digital
Retail & Hospitality
Commercial Equipment

Check out all our Monster brands at www.monstershop.co.uk
